

Johnson & Johnson

TECNIS  
Odyssey™ IOL

TECNIS Odyssey™ IOL

Precise vision.  
Every distance.  
Any lighting.<sup>1,4,5</sup>

TECNIS Odyssey™ IOL   
Overview

A high-quality, full visual range IOL<sup>\*1</sup>

Enhanced Tolerance to  
Refractive Error<sup>†2</sup> 

More consistent and  
reliable visual outcomes

Optimized  
Dysphotopsia Profile<sup>†3</sup> 

Improved night-vision quality

Outstanding Range  
of Vision<sup>\*\*14</sup> 

Continuous vision  
from far to near

TECNIS™ Best-in-Class  
Contrast<sup>††5-7</sup> 

Sharper vision in all lighting conditions

<sup>\*</sup>According to ISO 11979-7:2024, based on the clinical study of the parent IOL

<sup>†</sup>Compared to PanOptix® based on bench testing

<sup>†</sup>Compared to TECNIS Synergy™ OptiBlue™ IOL based on bench testing

<sup>\*\*</sup>Continuous 20/25 or better



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision

TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# TECNIS Odyssey™ IOL: Delivering Exceptional Patient Satisfaction

## Precise Vision

94%

satisfaction with overall vision without glasses.<sup>8</sup>

## Every Distance


96%

satisfaction with reading on a smartphone or tablet.<sup>9</sup>

## Any Lighting

92%

satisfaction with ability to see steps and curbs at night.<sup>9</sup>

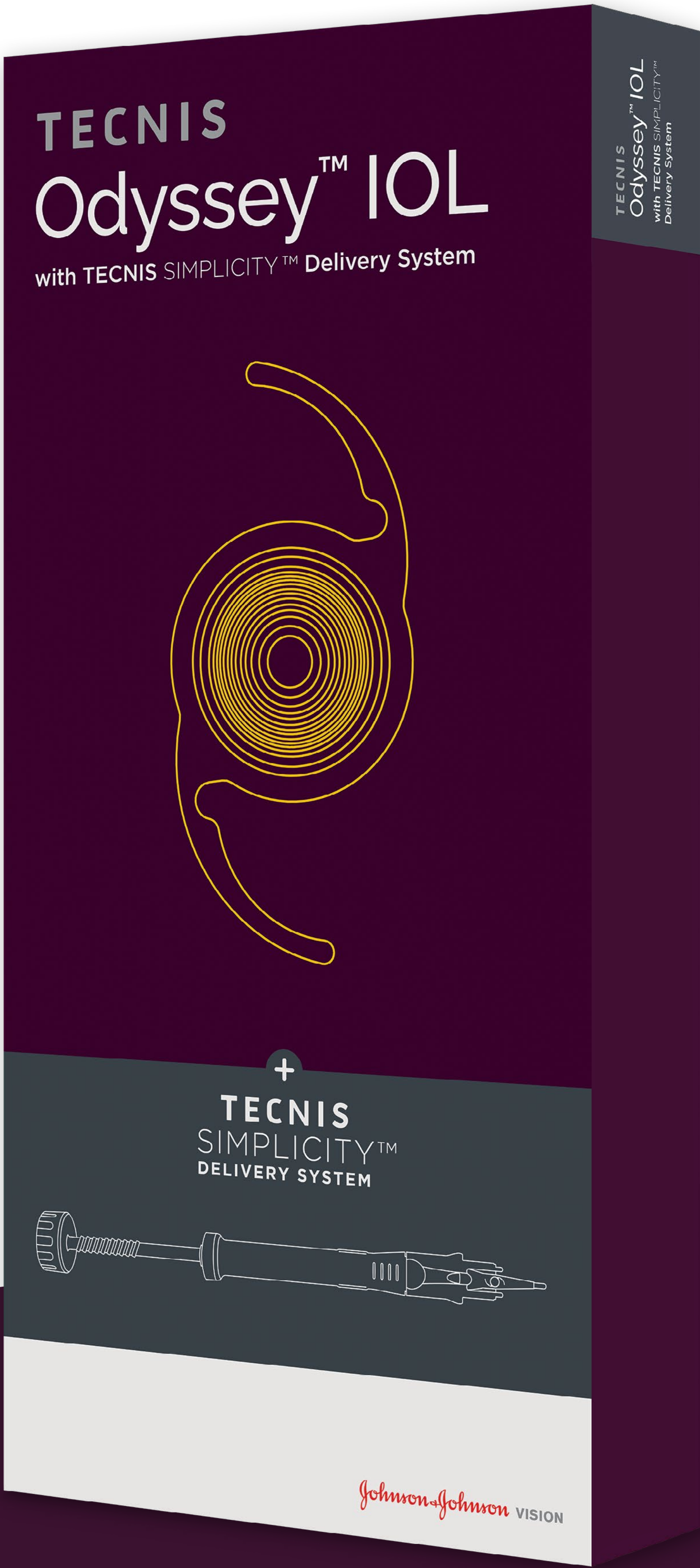

<sup>8</sup>Values rounded to the nearest 1%.  
<sup>9</sup>Based on 3-month postoperative data from a multi-center, observational clinical study in the U.S.



TECNIS  
Odyssey™

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision

TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# TECNIS Odyssey™ IOL: A Full Visual Range IOL\*1

Only TECNIS Odyssey™ IOL features an *all-new* Freeform Diffractive Profile.

| Technologies                                                                  | Features                                                                                                                                                                                                                                                                                        |
|-------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Improved with TECNIS Odyssey™ IOL:<br><b>Freeform<br/>Diffractive Profile</b> | <ul style="list-style-type: none"><li>➞ Enhances tolerance to refractive error†2</li><li>➞ Optimizes the dysphotopsia profile†3</li></ul>                                                                                                                                                       |
| <b>Hybrid Multifocal-EDOF Design</b><br><i>Best of both worlds</i>            | <ul style="list-style-type: none"><li>➞ Delivers a continuous, full range of vision:**4<ul style="list-style-type: none"><li>• Multifocal elements provide excellent distance and near visual acuities (VAs)</li><li>• EDOF elements provide continuous vision without gaps</li></ul></li></ul> |
| <b>Achromatic Technology<br/>&amp; TECNIS™ IOL Platform</b>                   | <ul style="list-style-type: none"><li>➞ Maximizes contrast, even in low-light conditions¶†5-6</li></ul>                                                                                                                                                                                         |



\*According to ISO 11979-7:2024, based on the clinical study of the parent IOL  
¶Compared to PanOptix® based on bench testing  
†Compared to TECNIS Synergy™ OptiBlue™ IOL based on bench testing  
\*\*Continuous 20/25 or better



# All-New Freeform Diffractive Profile

Digitally-optimized process enables new degrees of freedom in design.

**Digital Optimization:**  
Digitally fine-tuning the optical profile minimizes aberrations, effectively enhancing tolerance to residual refractive errors.<sup>2</sup>



**Advanced Design:**  
Custom-shaped echelettes and minimized step heights provide precise light control, minimizing halos, glare, and starbursts.<sup>3</sup>

**50%**  
smaller step heights\*



■ TECNIS Odyssey™ IOL

■ TECNIS Synergy™ OptiBlue™ IOL

*For illustrative purposes only*



\*TECNIS Odyssey™ IOL step heights are at least 50% smaller compared to the largest step height of TECNIS Synergy™ OptiBlue™ IOL



TECNIS  
Odyssey™

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision


TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform


Exceptional  
Rotational Stability

# Engineered for Surgeon Confidence

Freeform diffractive profile contributes to enhanced tolerance to residual refractive errors.\*§2,10

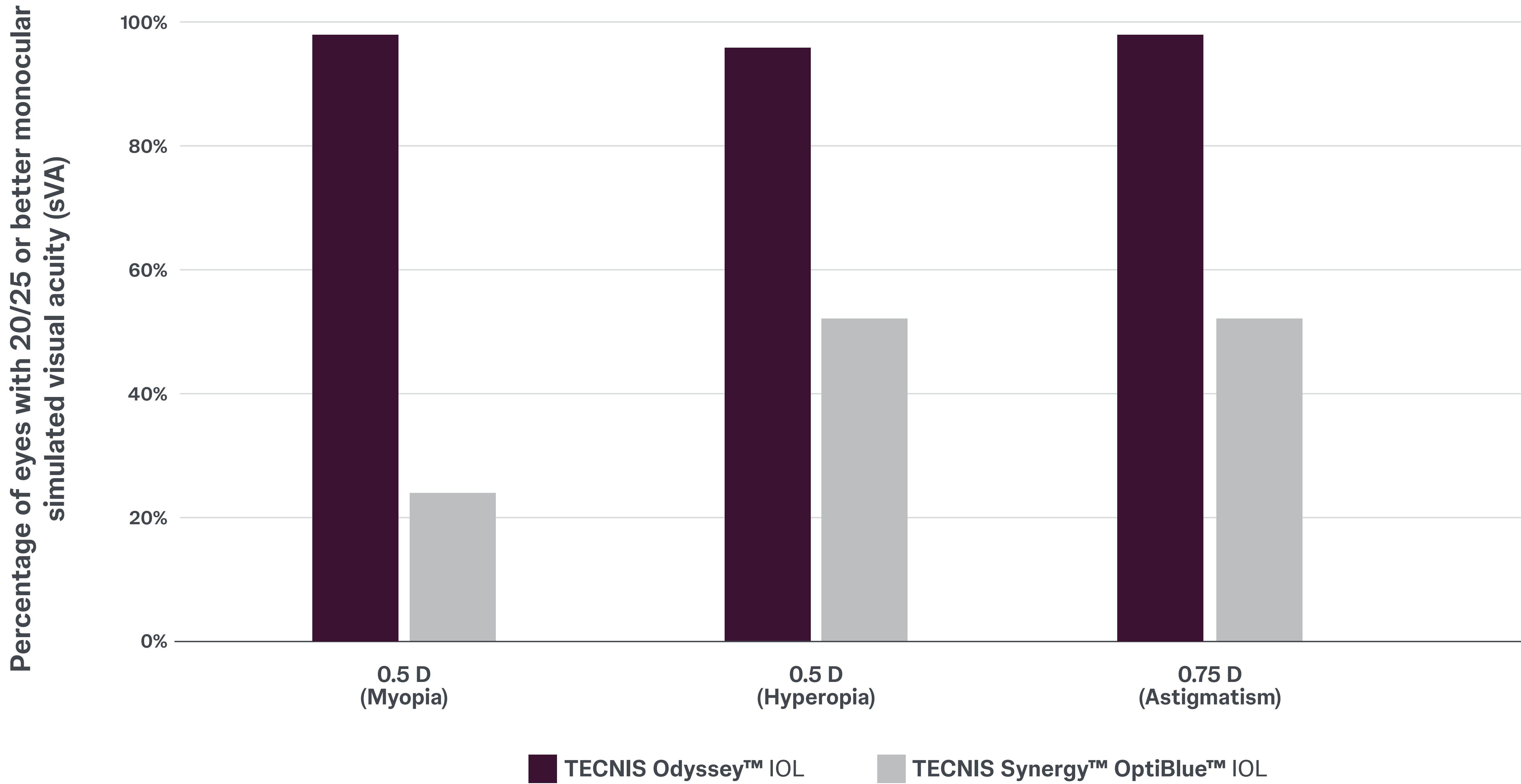

>95%  
of eyes achieved  
good distance  
VA, even under  
defocus.\*2,10

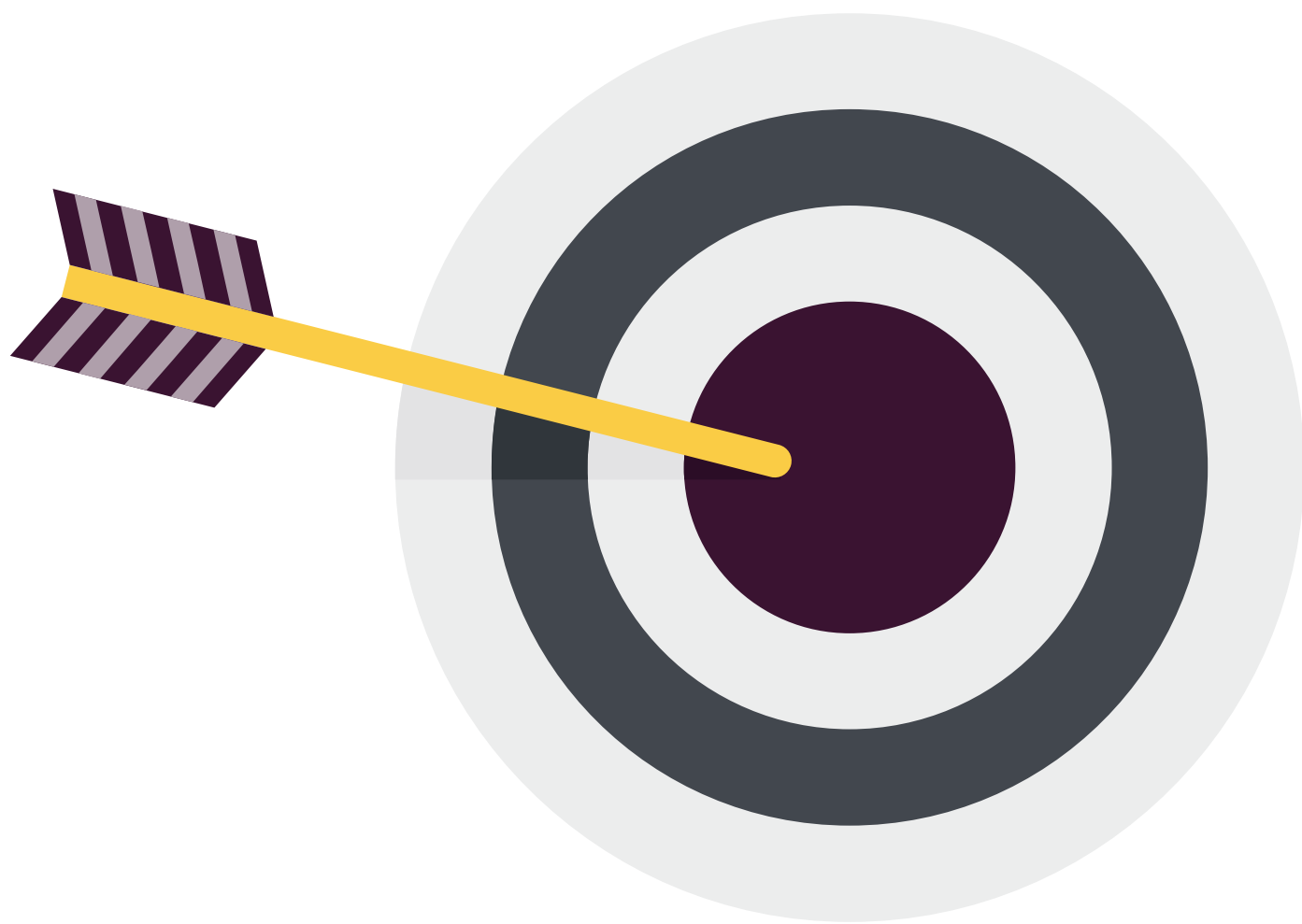

**Preoperative Guidance:**

- **Astigmatism Management:** Aim for  $\leq 0.5$  D of residual astigmatism postoperatively.
- **Target Plano to First Minus (slight myopia):** Targeting plus or beyond first minus is neither recommended nor necessary with this lens.



\*Based on bench testing.  
§Compared to TECNIS Synergy™ OptiBlue™ IOL



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision


TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# Optimized Dysphotopsia Profile

Freeform diffractive profile contributes to a **low incidence of bothersome visual disturbances**.<sup>11</sup>

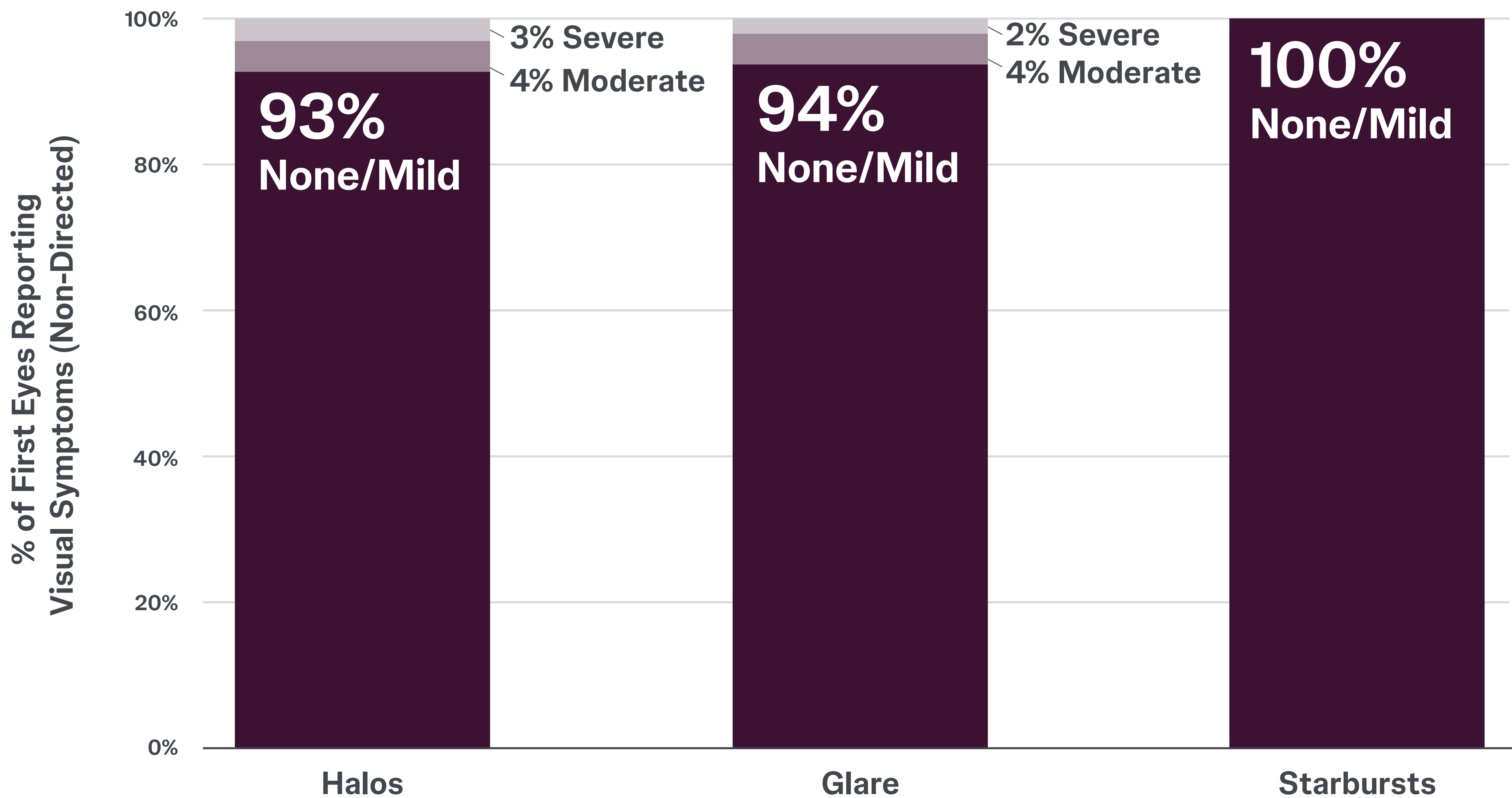

**93%**  
of patients either  
did not report any  
visual disturbances  
or reported mild  
halos or glare at one  
month post-op<sup>11</sup>



Values rounded to the nearest 1%  
Retrospective, multi-center, real-world clinical analysis of reported outcomes at 1 month post-operative visit, n=96. Symptoms reported without a specified severity level were classified as mild in the chart above.



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia  
Profile

Outstanding  
Range of Vision



TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# Minimized Visual Disturbances


TECNIS Odyssey™ IOL demonstrates a reduced halo profile.\*3,12

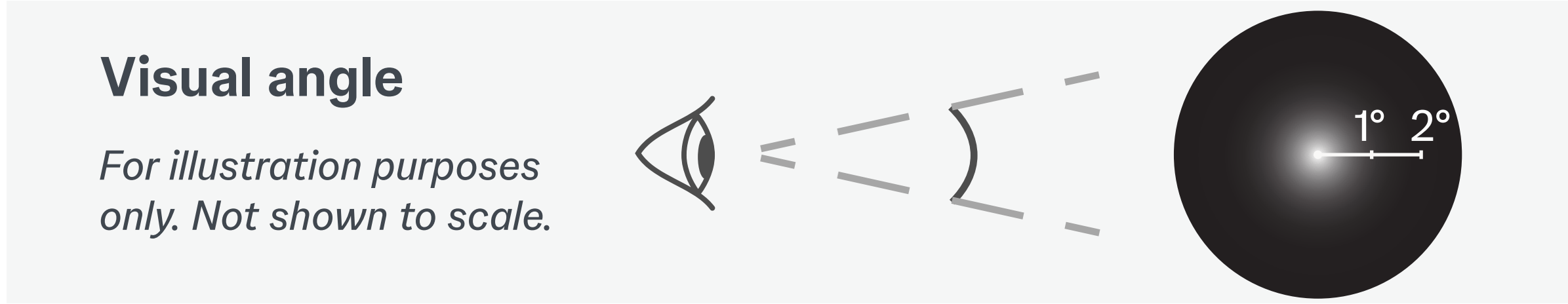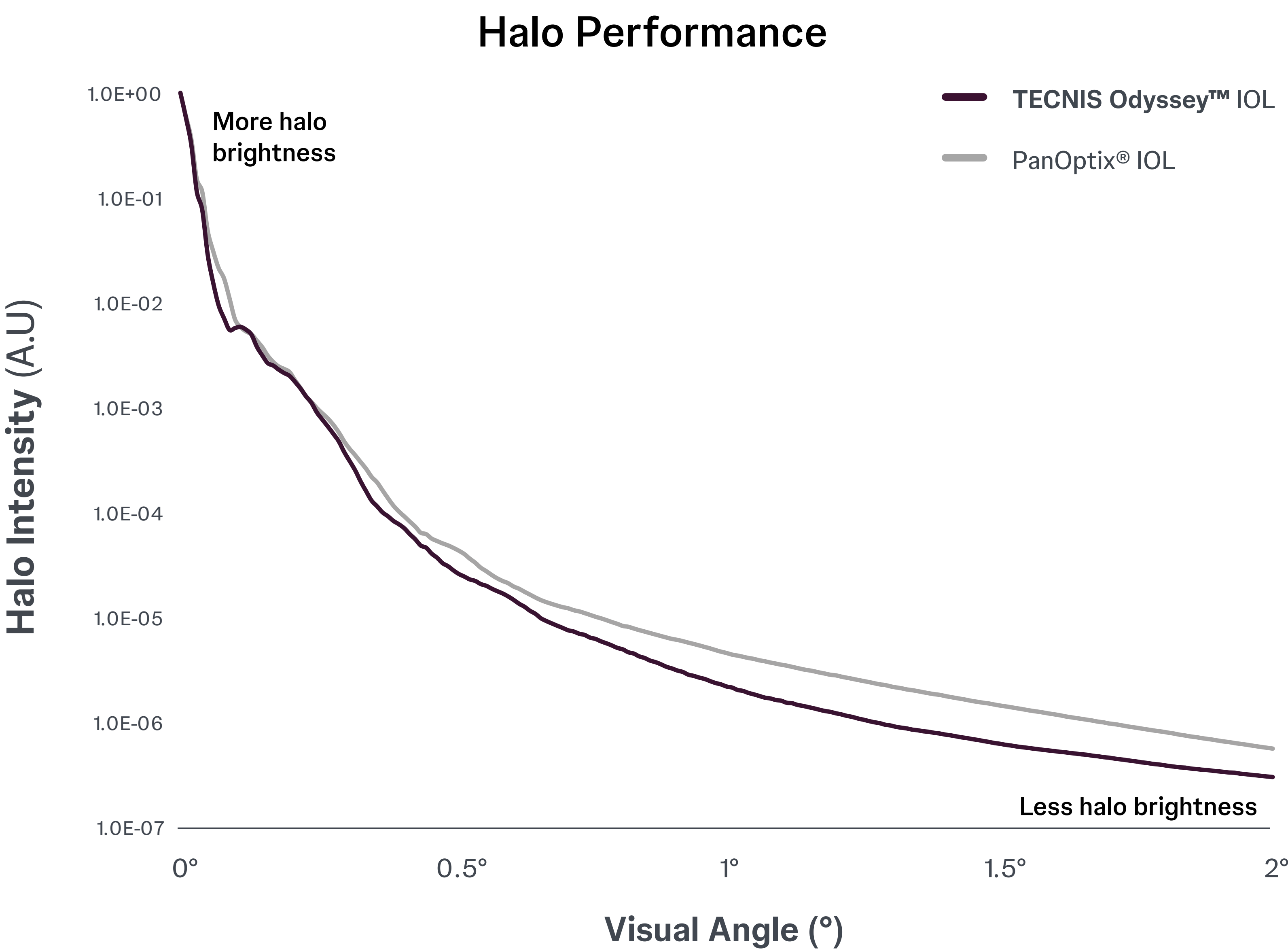


\*Compared to PanOptix® based on bench testing  
The third-party trademarks used herein are the trademarks of their respective owners.

## Optical Bench Halo Measurements (4.0-mm pupil size)<sup>12</sup>

TECNIS Odyssey™ IOL

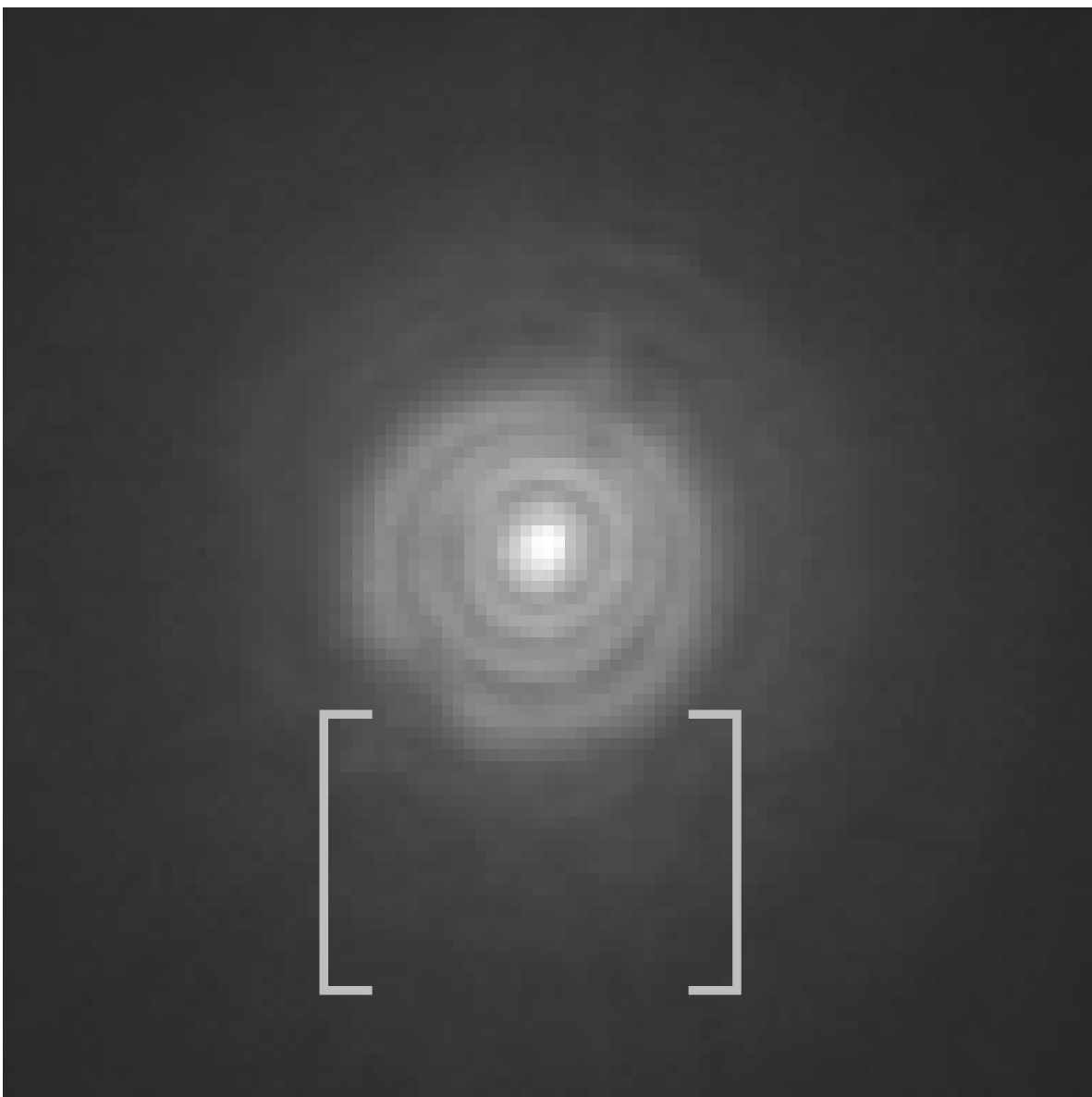

PanOptix® IOL

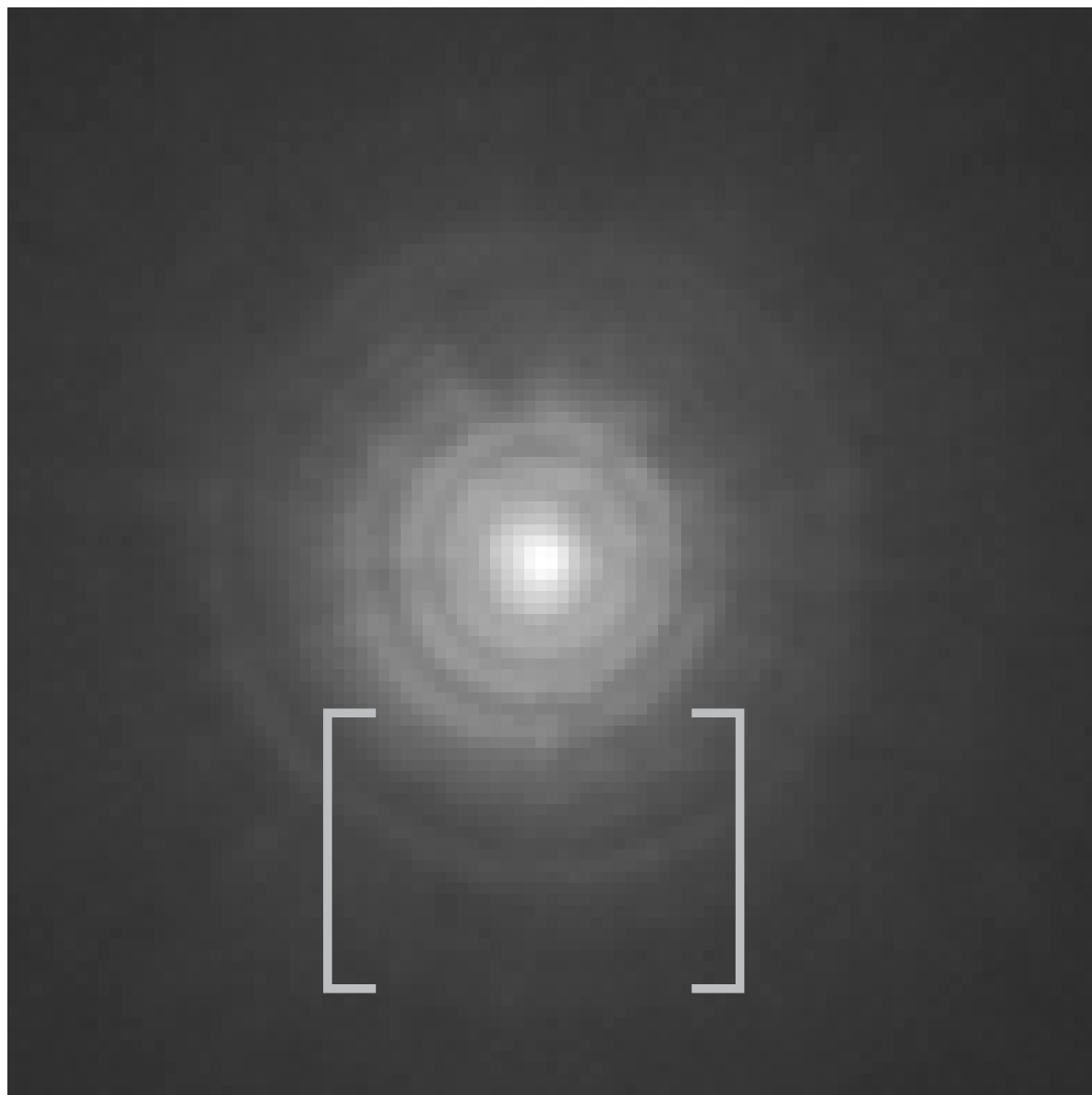

**TECNIS Odyssey™ IOL delivers lower halo intensity as the distance from the halo center increases.\*3,12**

In a pre-clinical lab assessment, **TECNIS Odyssey™ IOL** demonstrated lower intensity for most visual angles (0.67 to 2.0 degrees), indicating better halo performance than PanOptix® IOL. The light intensity values were normalized with reference to the peak intensity (center of the image), and a gamma correction of 0.1 was used. Presentation of the halos are shown in the images above. The assessment used the CDD, a research device designed to evaluate intraocular lenses. The device complies with ANSI standard Z80.35, section A.4.1.



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia  
Profile

Outstanding  
Range of Vision


TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# Outstanding Range of Vision

A continuous, full range of vision with better near compared to PanOptix® IOL.\*†‡

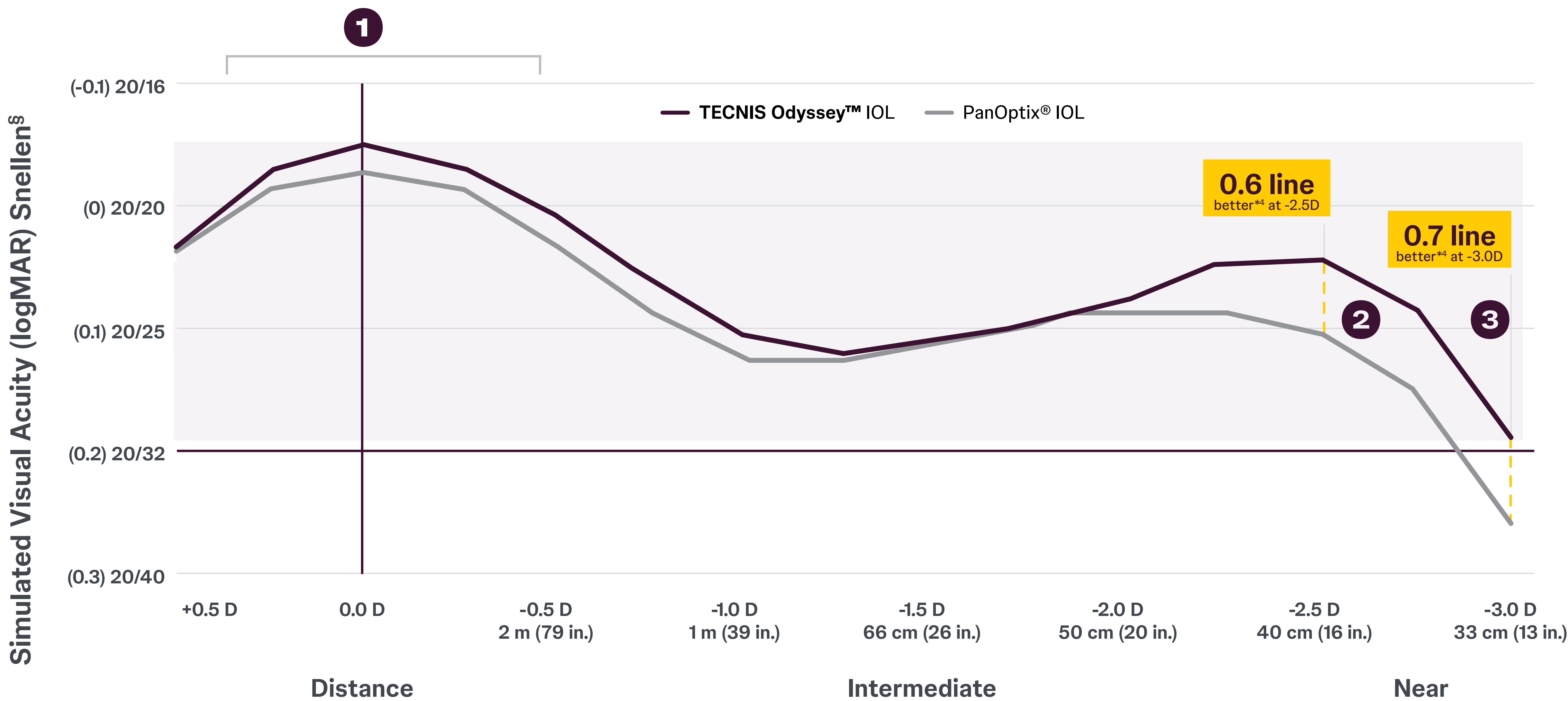

\*Based on bench testing compared to PanOptix®  
\*\*AUC=Area under the curve  
†continuous 20/25 or better  
‡above 0.2 LogMar (~20/32 Snellen) compared to PanOptix®

<sup>§</sup>Snellen VA was converted from logMAR VA. A Snellen notation of 20/20-2 or better indicates a logMAR VA of 0.04 or better

The third-party trademarks used herein are the trademarks of their respective owners.



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision


TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# TECNIS™ Best-in-Class Contrast

Superior low-light contrast compared to PanOptix® IOL.\*5, 7, 15

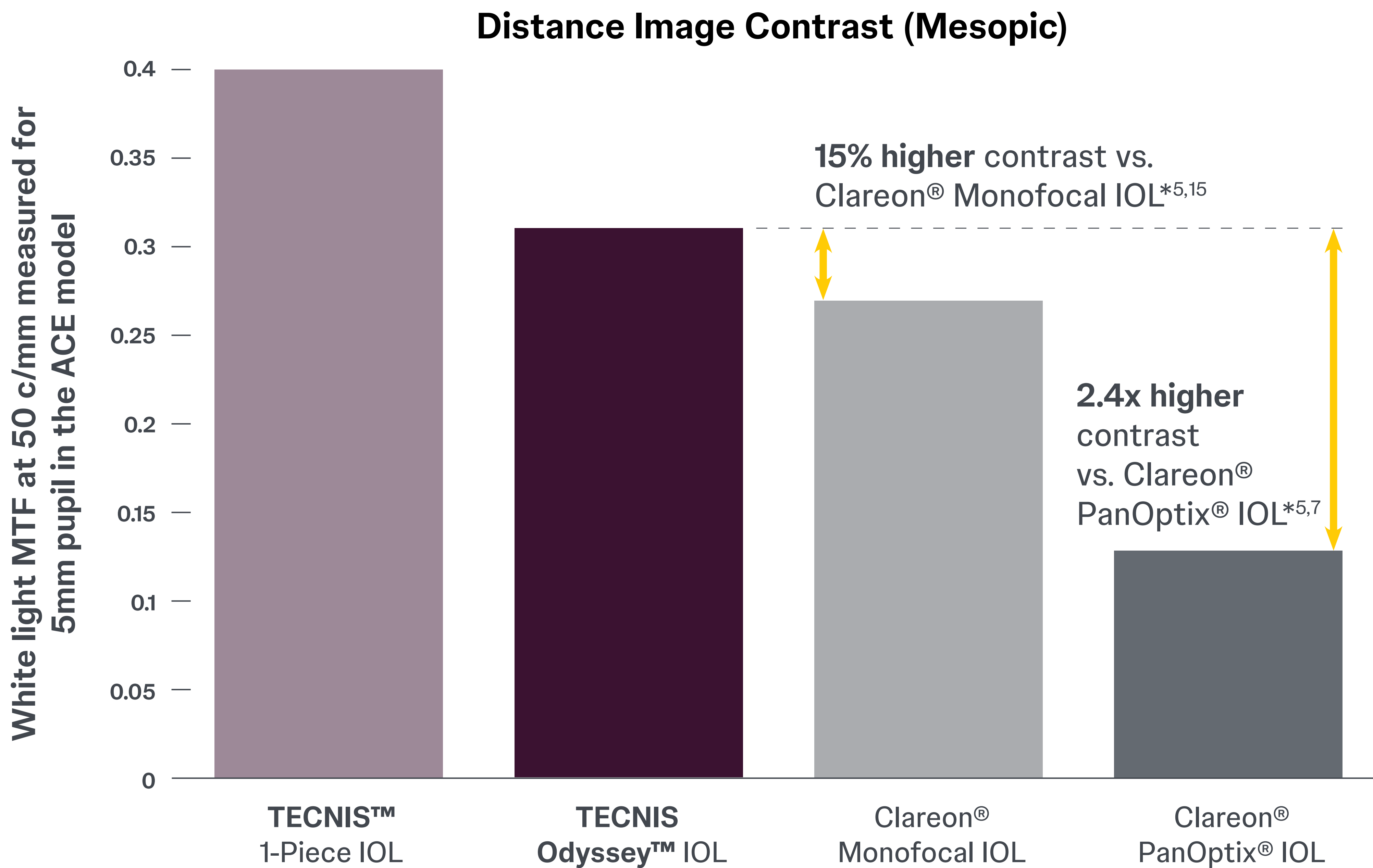

Modulation transfer function (MTF) measures the ratio of object contrast to image contrast. Higher MTF means more contrast transfer, enhancing the perceived image.

\*Based on bench testing  
Best-in-class compares to competitor full range of vision IOLs.  
The third-party trademarks used herein are the trademarks of their respective owners.



**TECNIS™ IOLs deliver high-quality vision** by minimizing primary optical aberrations.<sup>16,17</sup>

**TECNIS Odyssey™ features a proprietary Achromatic Technology**, further enhancing contrast by actively correcting chromatic aberration at all distances.<sup>18</sup>



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision

TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# High-Quality Vision Starts with TECNIS™ IOL

Experience the strength of a proven 20+ year platform.

## Outstanding contrast, day and night.\*16


TECNIS™ IOL material induces the least amount of chromatic aberration.

## Sharp quality of vision.17

TECNIS™ IOLs are one of the first and currently only IOLs that correct spherical aberration (SA) to essentially zero.19

## Sustained optical clarity and stability.20-22

TECNIS™ IOL design and material lead to low rates of PCO and capsular phimosis23

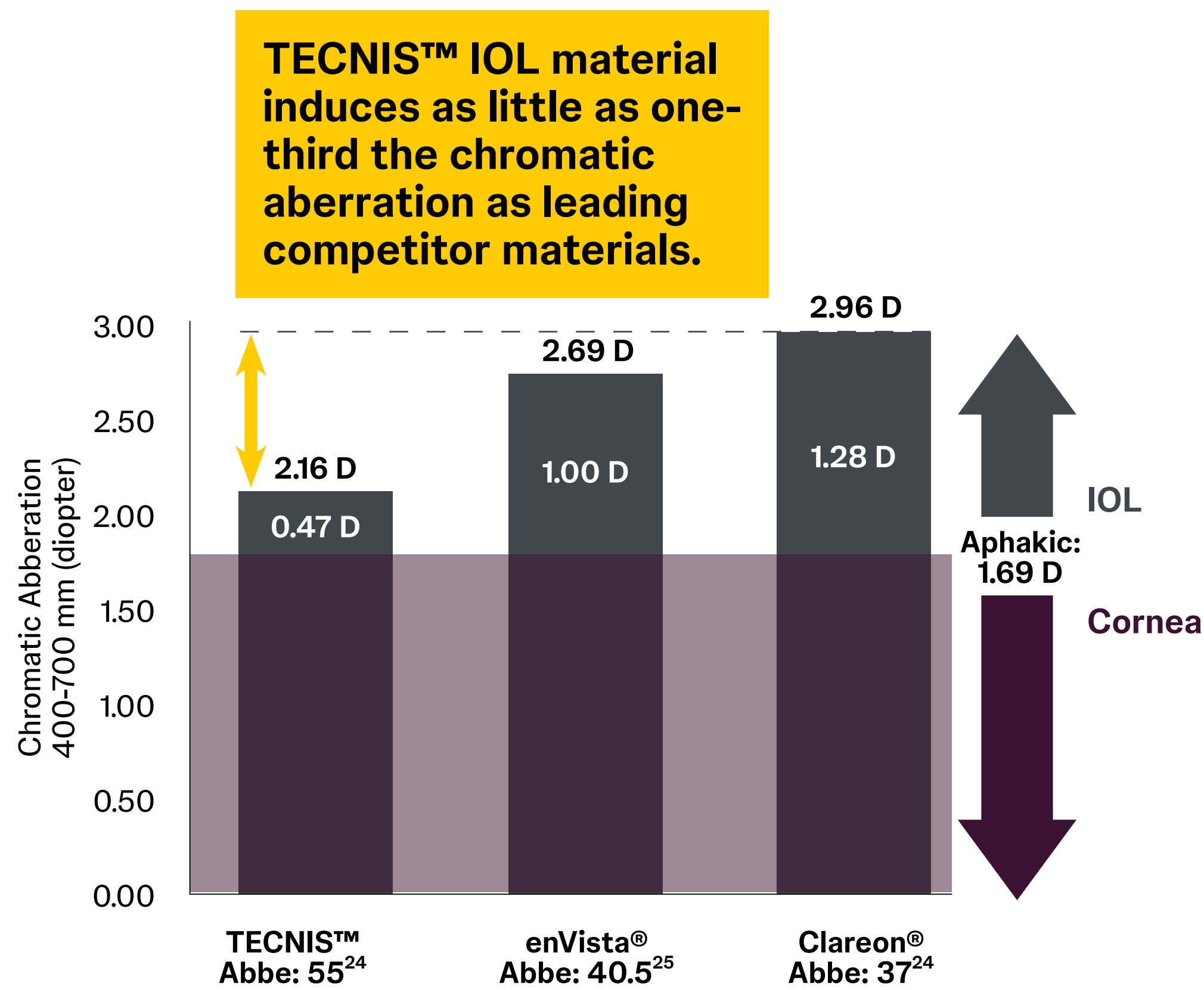

|                    | TECNIS™ | Clareon® | enVista® |
|--------------------|---------|----------|----------|
| Average Corneal SA | +0.27   | +0.27    | +0.27    |
| Lens SA‡           | -0.27   | -0.20    | 0.00     |
| Total Residual SA  | 0.00    | +0.07    | +0.27    |
| 20/20 ≠ 20/happy   | E       | E        | E        |

Increasing Asphericity†



TECNIS™ IOL material is not associated with glistenings.22



\*Compared to Clareon® and enVista® IOL platforms.  
†Images simulated using Zemike Tool, 6mm aperture, created by George Dai, PhD.  
‡SA correction of lens at corneal plane.  
Third party trademarks are the property of their respective owners.



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision



TECNIS™ Best-  
in-Class Contrast

TECNIS™  
Platform

Exceptional  
Rotational Stability

# Exceptional Rotational Stability

TECNIS™ Toric II IOLs have the lowest likelihood of needing repositioning\*\*26

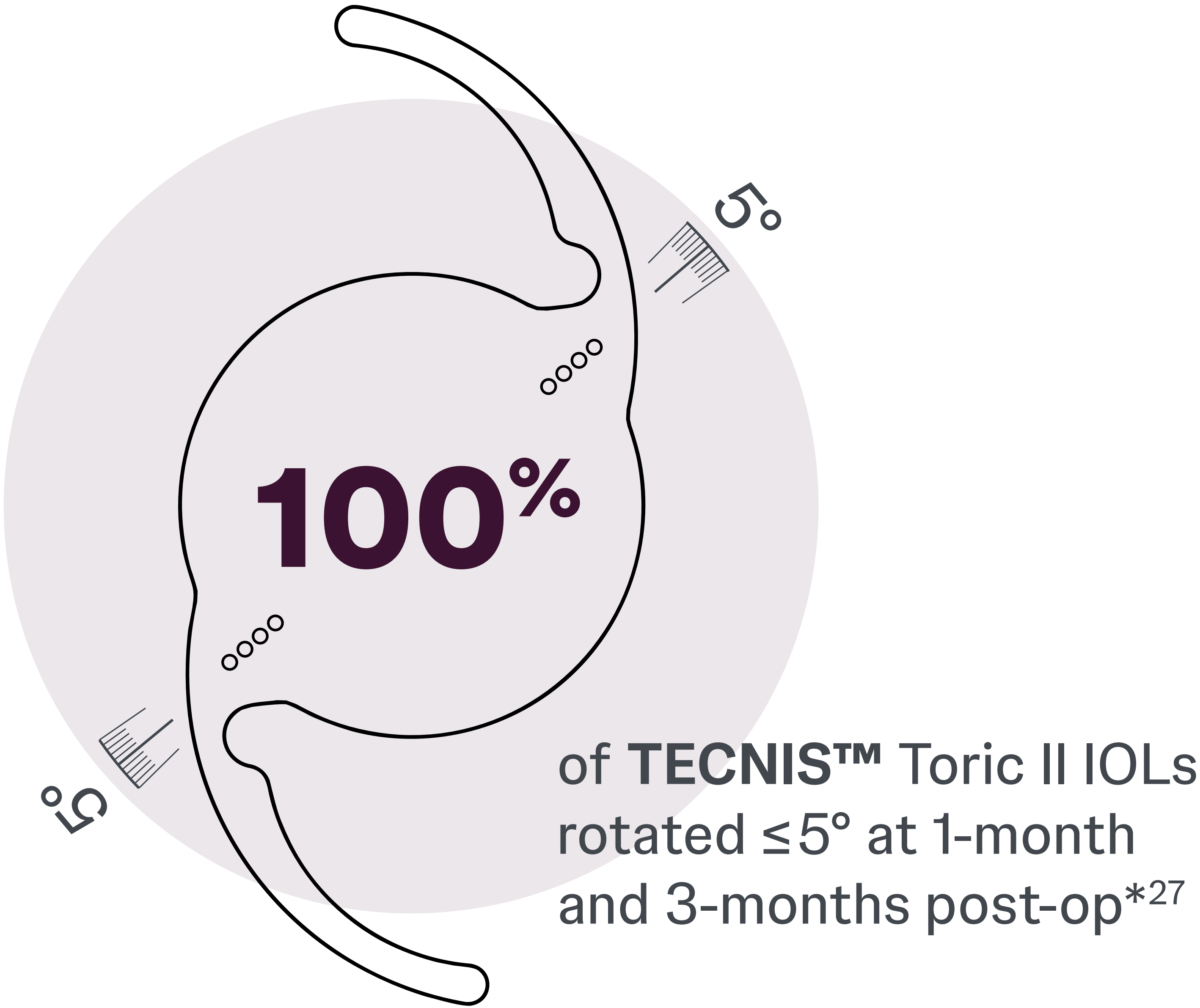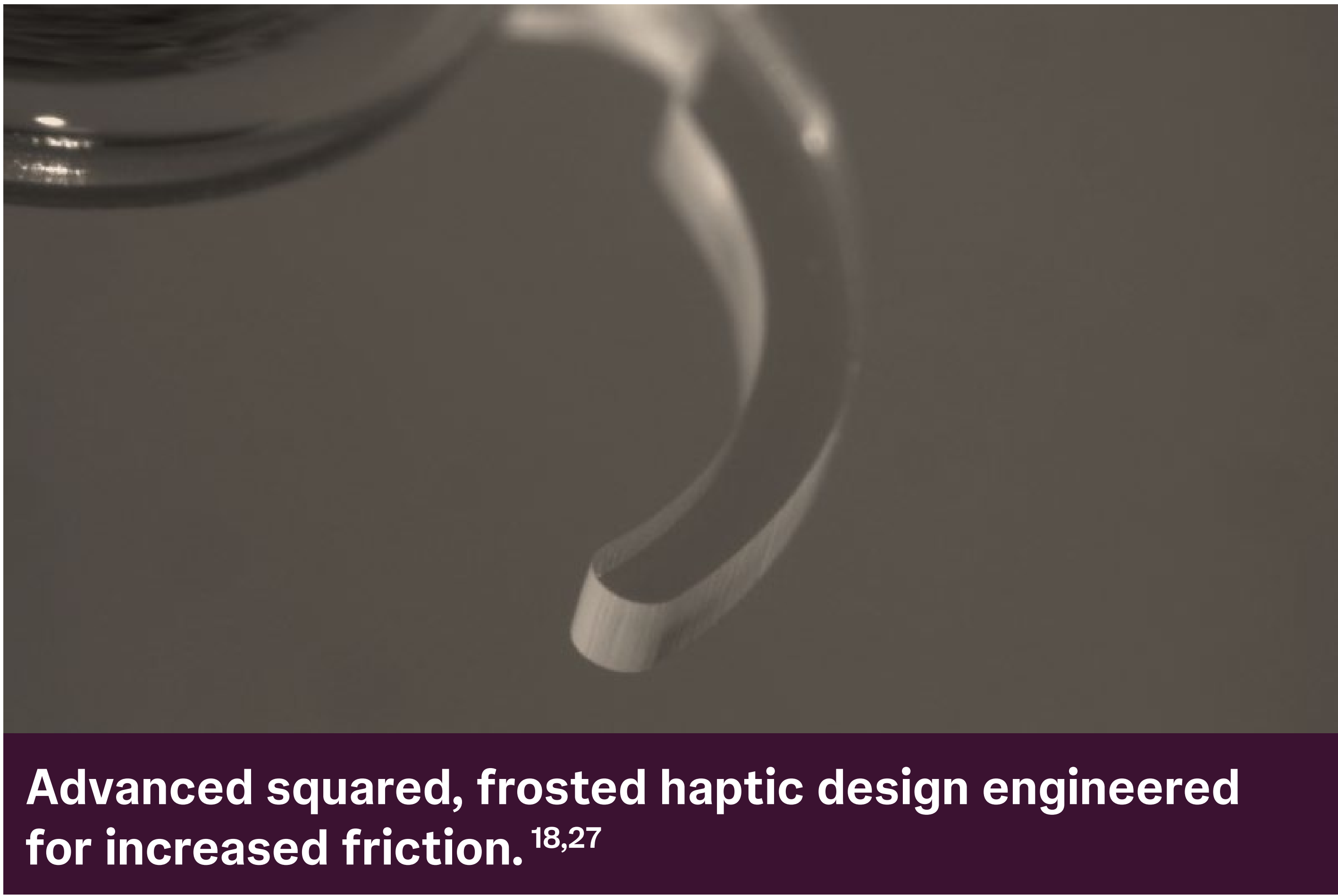

|                                                      |         |       |
|------------------------------------------------------|---------|-------|
| Outstanding mean rotational stability at post-op:*27 | Day 1   | 0.82° |
|                                                      | Month 3 | 0.94° |



\*Based on data from 200 eyes after 3 months postoperative follow-up in a postmarket prospective, multicenter, single-arm, open-label study of the TECNIS™ Toric II 1-Piece IOL conducted in the US. Outcomes differ from the pivotal investigation data in the product labeling and were collected using different measurement methods, study design and clinical conditions

\*\*Compared to AcrySof® Toric, enVista® Toric, and TECNIS™ Toric IOLs based on a retrospective chart review that looked at the rate of surgical IOL repositioning due to clinically significant IOL rotation comparing case records for 993 eyes implanted with TECNIS™ Toric II (n=308), AcrySof® Toric (n=362), enVista® Toric IOLs (n=270), or TECNIS™ Toric (n=53).



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision


TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

# Extraordinary Simplicity

Simplify lens delivery with TECNIS SIMPLICITY™ preloaded delivery system.

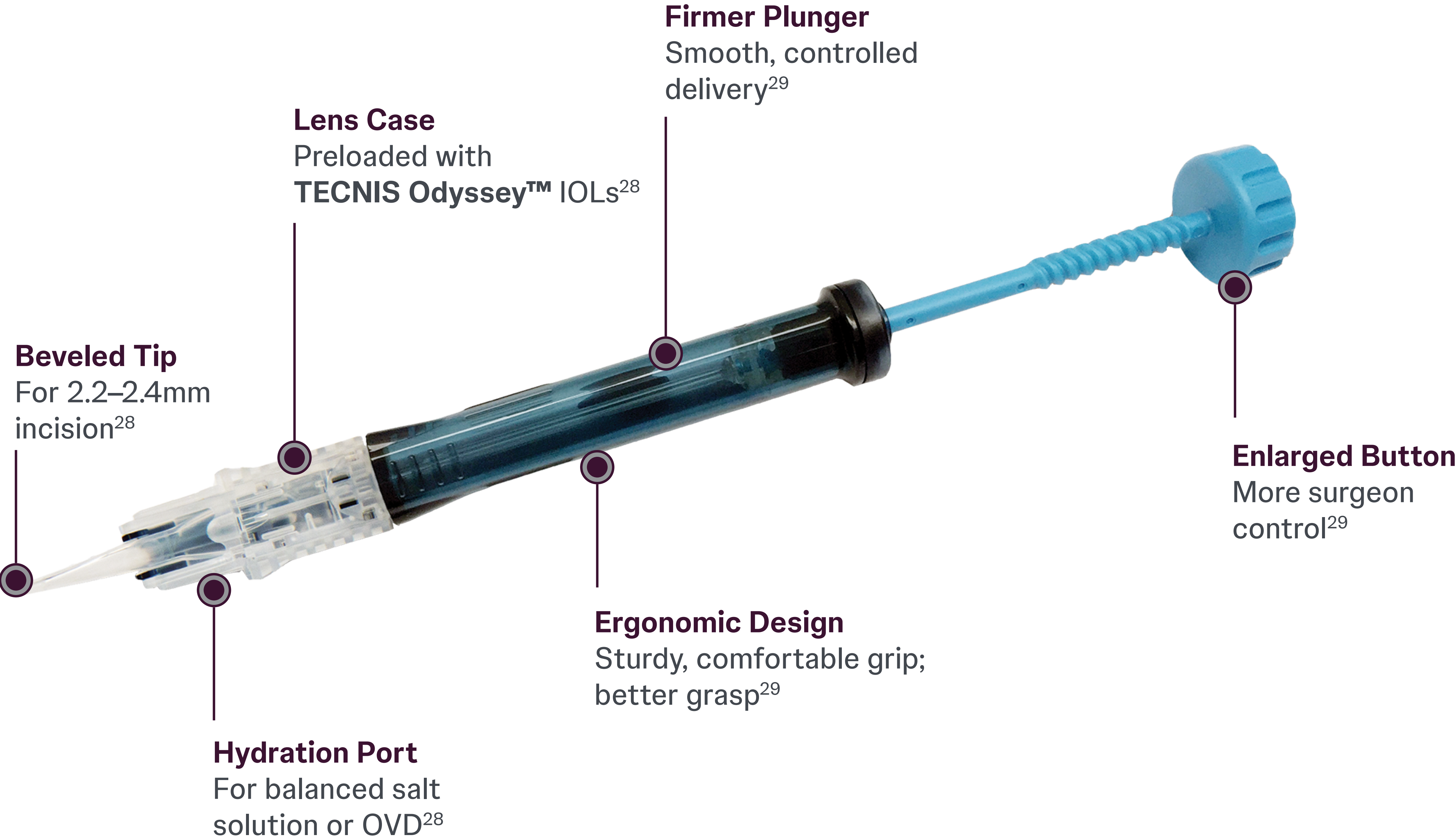

### Improved efficiency

Simple 3-step process: hydrate, advance and deliver<sup>29</sup>

### Enhanced safety

Minimize risk of infection associated with contamination<sup>28</sup>

### Smooth & controlled delivery


Designed to safely place the IOL into the capsular bag<sup>28,29</sup>

### Flexibility

Hydrate with balanced salt solution or OVD<sup>28</sup>

# Specifications<sup>18,28</sup>

| Optical Characteristics                      |                                                                                                                                                                  |        |        |                      |        |        |
|----------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------|--------|----------------------|--------|--------|
| Model numbers:                               | DRN00V                                                                                                                                                           | DRT100 | DRT150 | DRT225               | DRT300 | DRT375 |
| Cylinder Powers—IOL Plane:                   | N/A                                                                                                                                                              | 1.00 D | 1.50 D | 2.25 D               | 3.00 D | 3.75 D |
| Cylinder Powers—Corneal Plane:               | N/A                                                                                                                                                              | 0.69 D | 1.03 D | 1.54 D               | 2.06 D | 2.57 D |
| SE Powers:                                   | +5.0 D to +34.0 D in 0.5 diopter increments                                                                                                                      |        |        |                      |        |        |
| Diameter:                                    | 6.0 mm                                                                                                                                                           |        |        |                      |        |        |
| Center Thickness:                            | 0.7 mm (20.0 D)                                                                                                                                                  |        |        |                      |        |        |
| Shape:                                       | Biconvex, wavefront-designed anterior aspheric surface, proprietary posterior <b>freeform diffractive profile</b> to provide a full range of vision <sup>1</sup> |        |        |                      |        |        |
| Material:                                    | UV-absorbing hydrophobic acrylic with violet-light filter                                                                                                        |        |        |                      |        |        |
| Refractive Index:                            | 1.47 at 35°C                                                                                                                                                     |        |        |                      |        |        |
| Edge Design:                                 | ProTEC frosted, continuous 360° posterior square edge                                                                                                            |        |        |                      |        |        |
| Achromatic Technology:                       | Proprietary technology for chromatic aberration correction to enhance contrast                                                                                   |        |        |                      |        |        |
| Biometry*                                    |                                                                                                                                                                  |        |        |                      |        |        |
|                                              | Contact Ultrasound <sup>†</sup>                                                                                                                                  |        |        | Optical <sup>‡</sup> |        |        |
| A-constant:                                  | 118.8                                                                                                                                                            |        |        | 119.3                |        |        |
| AC Depth:                                    | 5.4 mm                                                                                                                                                           |        |        | 5.7 mm               |        |        |
| Surgeon Factor: <sup>30</sup>                | 1.68 mm                                                                                                                                                          |        |        | 1.96 mm              |        |        |
| Haptic Characteristics                       |                                                                                                                                                                  |        |        |                      |        |        |
| Overall Diameter:                            | 13.0 mm                                                                                                                                                          |        |        |                      |        |        |
| Thickness:                                   | 0.46 mm                                                                                                                                                          |        |        |                      |        |        |
| Style:                                       | C                                                                                                                                                                |        |        |                      |        |        |
| Material:                                    | UV-absorbing hydrophobic acrylic with violet-light filter                                                                                                        |        |        |                      |        |        |
| Design:                                      | TRI-FIX, Haptics offset from optics; 1-piece lens                                                                                                                |        |        |                      |        |        |
| Preloaded TECNIS SIMPLICITY™ Delivery System |                                                                                                                                                                  |        |        |                      |        |        |

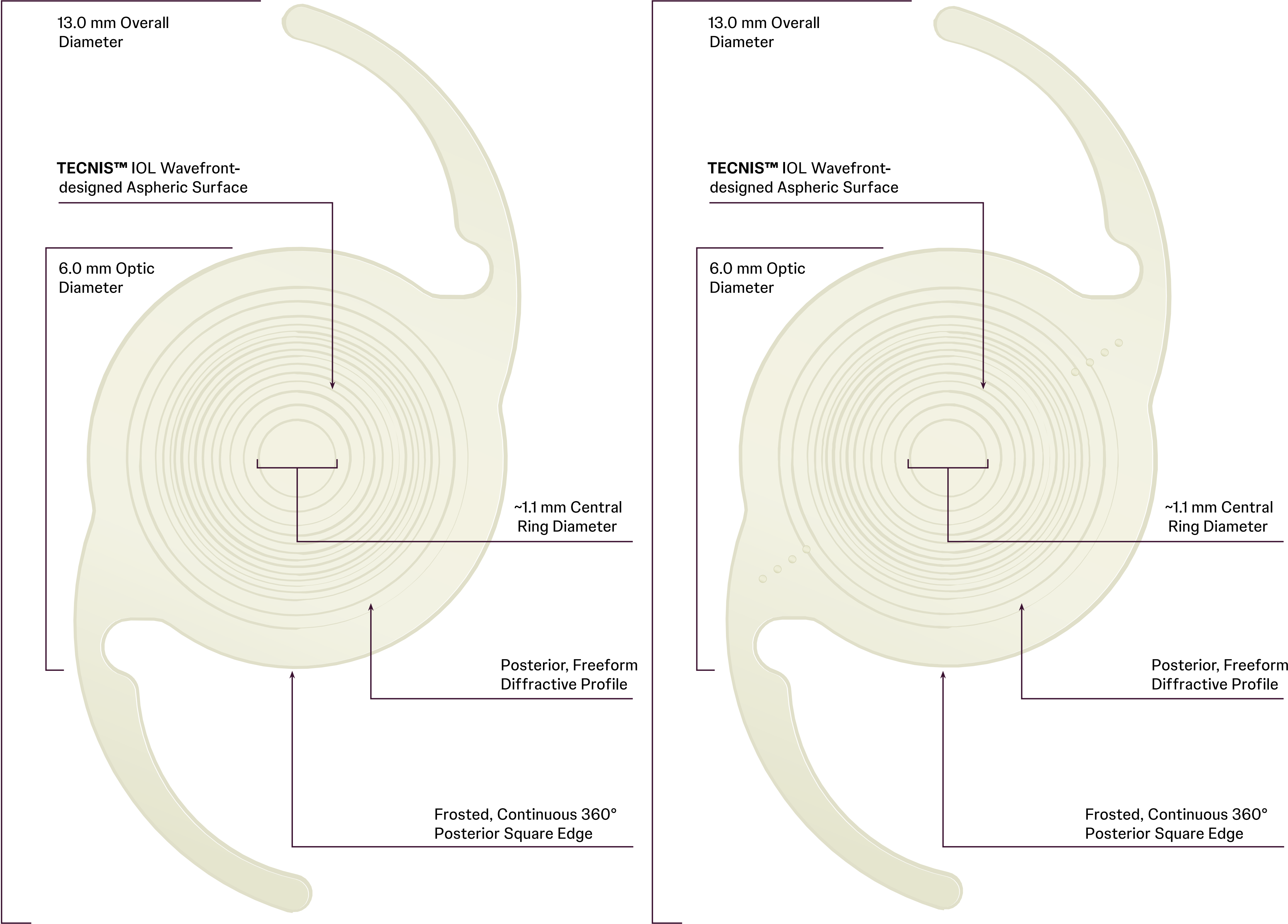

\* Values theoretically derived for a typical 20.0 D lens. Johnson & Johnson recommends that surgeons personalize their A-constant based on their surgical techniques and equipment, experience with the lens model and postoperative results.

† IOL constants have been theoretically derived for contact ultrasound.

‡ IOL constants have been derived from clinical evaluation results of the TECNIS™ 1-Piece IOL Platform.

For optimal results, utilize the **TECNIS™** Toric IOL calculator at **www.TecnisToricCalc.com** to determine the appropriate Toric model and power.



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision

TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability

References

1. 2024DOF4002 – **TECNIS Odyssey™** Full Visual Range IOL.

2. 2024DOF4003 – Simulated tolerance to refractive error (defocus) of the **TECNIS Odyssey™** IOL. Feb. 7, 2024.

3. 2024DOF4005 – Laboratory measurement of intensity of extended objects for halo illustration of **TECNIS Odyssey™** and PanOptix. Feb. 14, 2024.

4. DOF2023CT4023 – Simulated VA of the **TECNIS Odyssey™** IOL, Model DRN00V, and PanOptix® IOL, model TFNT00. Apr. 3, 2023.

5. DOF2023CT4007 – MTF of **TECNIS Odyssey™** lenses. 15 Mar 2023.

6. DOF2019OTH4002 – MTF of the **TECNIS Synergy™ OptiBlue™** IOL, and other lens models. 27 Mar 2019.

7. 2024DOF4033 – MTF of Clareon PanOptix lenses.

8. 2024DOF4027 - User Acceptability Evaluation of Pseudophakic Patients Previously Implanted with the **TECNIS Odyssey™** IOL. Patient Reported Spectacle Independence Questionnaire (PRSIQ). 11 June 2024.

9. 2024DOF4029 – User Acceptability Evaluation of Pseudophakic Patients Previously Implanted with the **TECNIS Odyssey™** IOL. Low Light Vision Questionnaire Outcomes. 11 June 2024.

10. 2024DOF4017 – Simulated tolerance to refractive error (cylinder) of the **TECNIS Odyssey™** IOL. May 13, 2024.

11. DOF2023CT4050 – Evaluation of Real-World Data on the Performance of the **TECNIS Odyssey™** IOL. Dysphotopsia Outcomes. 5 Feb 2024.

12. 2024DOF4016 – Laboratory images for halo illustration for the **TECNIS Odyssey™** IOL. May 3, 2023.

13. 2024DOF4015 – Simulated VA AUC of the **TECNIS Odyssey™** and PanOptix IOL.

14. DOF2023CT4056 – Minimum readable print size with **TECNIS Odyssey™** IOL. Oct. 30, 2023.

15. DOF2018OTH4004 – **TECNIS Eyhance™** and monofocal competitor IOLs MTF data. 7 Sep 2018.

16. DOF2018CT4007 – Chromatic aberration of the **TECNIS Symphony™** IOL. May 24, 2018.

17. **TECNIS™** 1-Piece IOL, Model ZCB00, DfU, Z311480E, current revision.

18. **TECNIS Odyssey™** Toric II IOL with **TECNIS SIMPLICITY™** Delivery System, Model DRT (DRT100-DRT375), DfU, Z311983E, current revision.

19. Eppig T, et al. Aberration Correction with Aspheric Intraocular Lenses. Intechopen.89361. 14 Oct 2019. 2024REF5180.

20. Nixon DR, Woodcock MG. Pattern of posterior capsule opacification models 2 years postoperatively with 2 single-piece acrylic intraocular lenses. J Cataract Refract Surg 2010;36(6):929-934. REF20140020.

21. Kugelberg M, et al. Posterior capsule opacification after implantation of a hydrophilic or a hydrophobic acrylic intraocular lens: one-year follow up. J Cataract Refract Surg 2006;32(10):1627–1631. REF2014CT0096.

22. Data on File 150 – Sensor Not associated with Glistenings – Literature Analysis. REF2014OTH0002.

23. Kahraman G, et al. Intraindividual comparison of capsule behavior of 3 hydrophobic acrylic intraocular lenses during a 5-year follow-up. J Cataract Refract Surg 2017;43(2):228-233. REF2018CT4194.

24. Zhao H, Mainster MA. The effect of chromatic dispersion on pseudophakic optical performance. Br J Ophthalmol. 2007;91(9):1225-1229. REF20140029.

25. Ligabue EA, Giordano C - Clinical Application of a New Hydrophobic Lens. - Introducing enVista - CRST Europe Nov-Dec 2011\_insert p. 5. 2024REF6159.

26. Hu EH. Repositioning Rates of Toric IOLs Implanted in Cataract Surgery Patients: A Retrospective Chart Review. Clin Ophthalmol 2023;17:4001-4007. 2024REF4080.

27. DOF2021CT4019 – Clinical Investigation of Rotational Stability of the **TECNIS™** Toric II Intraocular Lens. 20 Aug 2021.

28. **TECNIS Odyssey™** IOL with **TECNIS SIMPLICITY™** Delivery System, Model DRN00V, DfU, Z311982E, current revision.

29. Delta IOL Delivery System – Human Factors Validation Report. Dec. 12, 2018. REF2019CT4449.

30. Holladay JT. International Intraocular Lens & Implant Registry 2003. J Cataract Refract Surg. 2003; 29:176-197. REF2016CT0151.

For healthcare professionals only. Please reference the Instructions for Use for a complete list of Indications and Important Safety Information and contact our specialists in case of any question.

Third party trademarks are the property of their respective owners.  
© Johnson & Johnson and its affiliates 2025



TECNIS  
Odyssey™ IOL

Enhanced Tolerance  
to Refractive Error

Optimized  
Dysphotopsia Profile

Outstanding  
Range of Vision

TECNIS™ Best-  
in-Class Contrast

TECNIS™ IOL  
Platform

Exceptional  
Rotational Stability